

Login Screen

| Main | Info | Preferences | Work List | Admin | Archive | Reports | |
|---|----------|--|--|---|--|--|------|
| Home | Sign Off | | | | | | Help |
| Sign on to eRAD PACS User ID Password Sign On Forgot Passw | ord? | Florida Radiology Cons service will a The material permitted on Approved per with other ind passwords. | ultants is pleased to offer allow you to view your pati viewed contains confidential m this site. Users must gain acco sons will be given a user name lividuals. It is the user's respo | Internet services to enhan ient's images and reports. I nedical records. Only those perse ess to the Internet services by or and password, which are the re nsibility to protect and maintain Service: Swamp Cabbage Arch | ce the clinical care that By logging in, you agree ons approved access to these ompleting an Internet Access isponsibility of the user and r the security and confidentia | your patient receives. This to the following: records are Request form. to to be shared lity of these | |

Enter User ID and Password

Work List

| Main | Info | Preference | es | Work Li | ist | Admin | Archive | | Reports | |
|--------------------|------------|--------------|-----|---------------|------------------|-----------|--------------|----------|------------------|-------------------|
| DefaultList | OtherLists | Folders | | Schedul | ing Co | rrections | Group Orders | Dov | mloadViewer | Help |
| gquidera : default | | : [2] | | | I (| • | | | | Page 1 |
| Batch 🎬 🗑 🗭 🕫 🖉 | Date 🛆 | Patient Name | DoB | Report Status | Refer. Physician | | Description | Modality | Institution Name | Reading Physician |
| Select All None | <i>8</i> 4 | 26 | 293 | Final | 293 | | <i>M</i> 3 | 24 | * | <i>8</i> % |

• To Search for a patient click on the binoculars located under the "Patient Name" heading.

Search Box

| Fields | Settings | | | | | | | |
|--------------|-----------------------------------|--------|---------|-------|------|-------------|----------------|--------|
| Patient Name | E3 HH Pref: E3 HH ² | First: | Middle: | Last: | Suf: | | | |
| | | | | | | Apply Alone | Apply Combined | Cancel |
| | | | | | | | | |
| | | | | | | | | |

- Input last name and first initial of first name.
- Click on "Apply Combined" to search for patient's exam ordered by your group.
- Click on "Apply Alone" to search for patient's exam ordered by a physician outside of your group.
- Note: Pressing enter will default to "Apply Combined".



Work List

| Main | Info | Prefe | rences | Wor | k List | Admin | Archive | I | Reports | |
|--------------------|-------------------------|--------------|----------------|---------------|------------------|-------------------|--------------|----------|------------------|-------------------|
| DefaultList | OtherLists | Fo | lders | Sche | eduling | Corrections | Group Orders | Dowr | nloadViewer | Help |
| gquidera : default | | ÷ | | | | | | | | Page 1 |
| Batch 🕼 🗊 🗭 👁 🖉 | Date 🛆 | Patient Name | DoB | Report Status | Refer. Physician | Des | scription | Modality | Institution Name | Reading Physician |
| Select All None | 盎 | # \$ | #4 | Final | # % | | # \$ | #4 | # h | # |
| ▫▫ਫ਼₽₽₽ | Jun 05,2013 14:51:36 | Doe, Jane | Aug 30,1945 | Final | Dr. Feel Good | 3i LEFT HIP/PAIN/ | NO PREV | CR | FRC Barkley | Negin, Geoffrey |

- Click on "Final" adjacent to patient's name to pull up report.
- Click on "DefaultList" to go back to practice work list

Report Page

| Main | Info | Preferences | Work List | Admin | Archive | Reports | |
|-------------|--|-------------|--|---|---|--|------|
| DefaultList | OtherLists | Folders | Scheduling | Corrections | Group Orders | DownloadViewer | Help |
| 0 | FLORIDA RADIOLOGY CONSULTANTS xtraordinary Service. Excellent Car | 6 | 6311 South Pointe BN 63 Barkley Circle • Su 15000 Shell Point Blvo 1699 Periwinkle Way 1661 Estero Boulevar Medical Records: 239. | vd. • Suite 600 • Ft. Myers, ites 100 & 101 • Ft. Myers, d • Ft. Myers, FL 33908 • Sanibel, FL 33957 d • Fort Myers Beach, FL 3: .938.3511 * Scheduling: | FL 33919 FL 33907 3931 239.938.3535 | Print Friendly A Back to List Open Edit Report Dictate Fax | |

- Click "Print Friendly" to print report. (default printer automatically selected)
- Click "Open" to view images.
- Click "DefaultList" to go back to work list.
- Click "Back to List" to go back to search list.

If you have any login issues or need assistance using the PACS software, please contact our office at 239-938-3500.